




GIMA

PROFESSIONAL MEDICAL PRODUCTS

GONIOMETRO DA 7" ROBINSON 180° **180° ROBINSON GONIOMETER 7"** **GONIOMÈTRE 7" 180° ROBINSON 180°** **GONIÓMETRO ROBINSON 180° - 7"**

Manuale d'uso - User manual - Manuel de l'utilisateur - Guía de uso

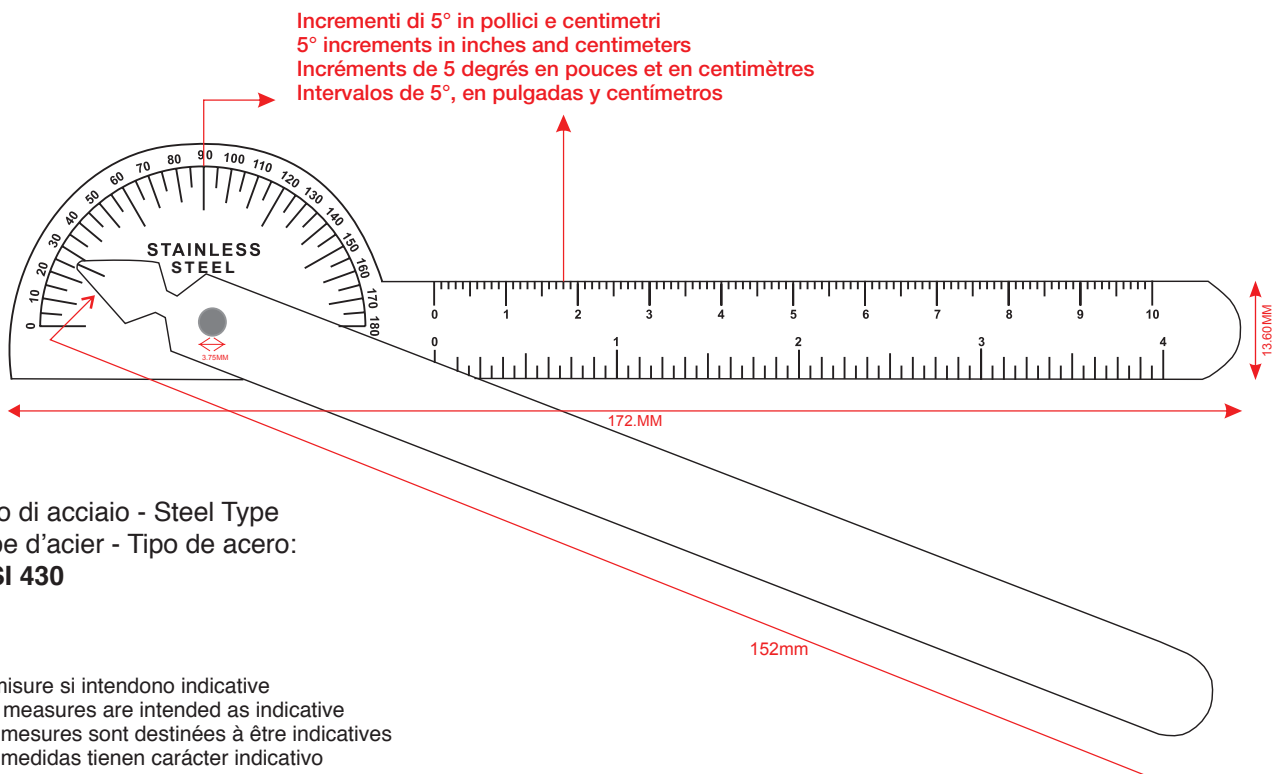

REF 24902

Importato da / Imported by / Importé par / Importado por:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

Made in Pakistan



GB - INSTRUCTIONS OF USE

Goniometers measure range of motion of finger joints (metacarpophalangeal and interphalangeal) and other small joints including toes. A goniometer measures range-of-motion (ROM) of various joints.

The 180° head is marked in 5° increments.

Arm measures in inches and centimeters.

Tension between the arms is controlled by the thumb knob.

INDICATION OF USE & PRECAUTIONS





Those devices are designed for professional use by qualified operators with necessary skills.

The use of the product involves direct contact with patient's skin; always wear protective gloves, do not use on dull skin, handle carefully to avoid accidental abrasion.

Clean and sterilize before each use; use neutral non-aggressive disinfectant solution (PH7), do not use solutions containing chloride ions or high alcohol content.

This product is intended to provide easy-to-read indicative measurements.

Simboli / Symbols / Symboles / Simbolos

	<p>IT Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB Caution: read instructions (warnings) carefully FR Attention: lisez attentivement les instructions (avertissements) ES Precaución: lea las instrucciones (advertencias) cuidadosamente</p>		<p>IT Conservare al riparo dalla luce solare GB Keep away from sunlight FR À conserver à l'abri de la lumière du soleil ES Conservar al amparo de la luz solar</p>	<div style="border: 1px solid black; padding: 2px; display: inline-block;">LOT</div>	<p>IT Numero di lotto GB Lot number FR Numéro de lot ES Número de lote</p>
	<p>IT Leggere le istruzioni per l'uso GB Consult instructions for use FR Consulter les instructions d'utilisation ES Consultar las instrucciones de uso</p>		<p>IT Conservare in luogo fresco e asciutto GB Keep in a cool, dry place FR À conserver dans un endroit frais et sec ES Conservar en un lugar fresco y seco</p>	<div style="border: 1px solid black; padding: 2px; display: inline-block;">REF</div>	<p>IT Codice prodotto GB Product code FR Code produit ES Código producto</p>