FABBRICA ARTICOLI
MEDICO SANITARI
MEDICAL SANITARY ITEMS

TITANOX s.r.l.

Via Canove, 2/a - Loc. Canove de Biazzi
26038 Torre de Picenardi (CR) Italy
Tel. +39 0375 394065 (r.a.) - Fax +39 0375 394067
Info@ titanox.it - www.titanox.it VAT IT 00679310193
Reg.Imp. CR Cod.Fis. e P.IVA 00679310193
Pos.Comm. Estero CR 005295 - R.E.A. CR n. 105063

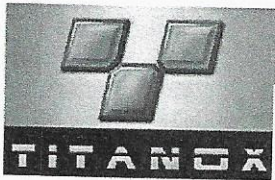
**ISTRUZIONI D'USO/USER INSTRUCTIONS/INSTRUCTIONS D'EMPLOI
POLTRONA A ROTELLE / WHEEL CHAIR/ FAUTEUIL ROULANT
CODICE/ITEM CODE/REFERENCE**

T5000

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Structure made of epoxy painted oval and round steel tube.
Backrest adjustable by gas pump.
Legrest manually adjustable.
Metal footrest manually extractable.
Armrests with articulations, manually adjustable in height.
Padding made of high density polyurethane foam, fireproof class 1 IM.
Covering made of washable, fireproof, plastic material (leatherette) - fireproof class 1 IM.
Back push handle.
Mattress overall dimensions: 166x55 cm.
Height from floor to seat: 56 cm.
Four wheels diam. 100 mm, 2 of which are directional and 2 with brake.
Supplied with chart holder support and blanket tray complete with fixing belts to fix oxygen bottle.
****** The item is supplied already assembled.**

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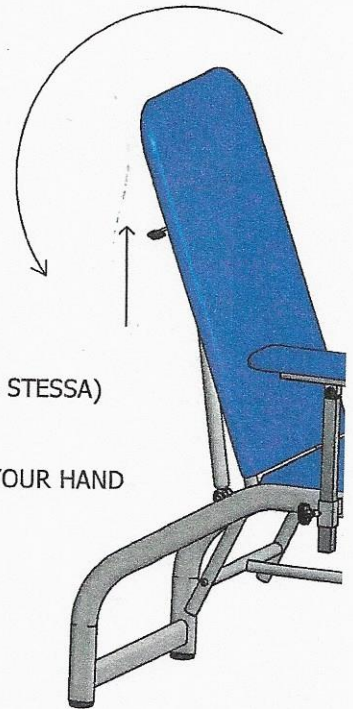
HANDLE: ALWAYS CHECK THE HANDLE NOT BE LOOSE,
IN CASE IT IS LOOSEN, TIGHTEN IT

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PUSH THE LEVER UPWARD, TO LOWER THE BACKREST
(N.B. HELP THE POSITIONNING OF THE BACKREST, BY PUSHING SOFTLY WITH YOUR HAND
IF A PERSON IS LYING ON THE COUCH)

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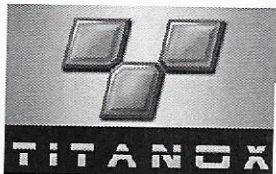
PUSH THE LEVER TO THE LEFT TO LIFT THE BACKREST
(N.B. HELP THE POSITIONNING OF THE BACKREST, BY PUSHING SOFTLY WITH YOUR HAND
IF A PERSON IS LYING ON THE COUCH)

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Cleaning Notice

The manufacturer company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

To clean the product, use only a clean cloth imbued of water and then dry with a clean and dried cloth.

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasiveness of the plastic surfaces and rusting of metal surfaces.

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