

- Proceed to apply the electrode to the patient as described in the instructions supplied by the electrode manufacturer.

NB: make the connections with the equipment turned off.

- If it is not required for other use, leave the adapter inserted in the cable-head even when the equipment is not in use.
- To effect the disconnection of the adapter from the cable head, exert opposing tractive force between the adapter cable shaft and the cable-head shaft to which the adapter is connected.

WARNING

- **The product is supplied non-sterile.**

- Do not make connections or disconnection when the equipment is turned on.
- The equipment for electro-stimulation and the electrodes used with this device must all be compliant with the regulations in force.

NB: Only ever use equipment designed for electro-stimulation at low current intensity.

- The device must be connected and started up by qualified personnel.

CLEANING

For cleaning detergent solutions (non-ionic surface-actives) and disinfectant solutions (components with quaternary ammonium of 0.1 - 0.2%) must be used. For the use of these products follow the relative instructions. After cleaning rinse under running water.

NB: Dry well before use.

LIFE OF PRODUCT

The life of the products depends on the number of uses and on maintenance type. The products must be immediately destroyed in case of deterioration of external insulation and connections and/or oxidation of connectors or plugs.

NB: Snap connection models must be immediately destroyed in case electrode doesn't stick perfectly to cable connection.

STORAGE

The package must be stored at a temperature between 0°C and 50°C and relative humidity between 20% and 80%.

GUARANTEE - NOTICE

FIAB guarantees that the product complies with Directive 93/42/EEC and has been manufactured according to the procedures of FIAB Quality System certified ISO 13485. No responsibility may be ascribed to the producer who shall not be held liable for medical costs, direct or indirect damage due to lacking function or malfunction of the above products, when used differently from the instructions for use. We recommend to report opportunely any malfunction or defect of the product to FIAB Quality Assurance Service.

DISPOSAL

Waste deriving from health structures must be disposed of according to the regulations in force. While the disposal of refuse deriving from civil residences does not require observance of any particular precautions of the regulations in force regarding urban refuse, it is recommended that the product be treated with disinfectant solutions (chlorine oxides, sodium hypochlorite, etc.) before being disposed of.

CAUTION: U.S.A. FEDERAL LAW RESTRICTS THIS DEVICES TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

EN

ADAPTERS FOR CONNECTOR CABLES FOR ECG AND ELECTROSTIMULATION

DESCRIPTION AND INDICATIONS

These articles have been designed to enable the adaptation of electrode connector cables which, given the type of connection they are equipped with, would not normally be compatible with the basic equipment cables. The FIAB adapters can be used for the connection of both disposable and multi-use electrodes to equipment for electro-stimulation and for electro-cardiac recording.

INSTRUCTIONS FOR USE

Carefully check packaging, cables and connection plugs. Do not use the product in case of visible damages or defects.

- Having checked the compatibility of the connections, insert the cable connector in the chosen adapter.